

NEW PRODUCT!

16% Hydrogen Peroxide Teeth Whitening Gel



Syringes & Kits



Procedure



What you need to know



Remineralize & Rehydrate



Waiver

Finally



High Profits

Each 3ml syringe comes with
3 service kits.

3ml Syringes

Up to 1ml will be used per
client.





Procedure

**Follow your
procedure for
blended gel.
2-3, 15 minute
rounds.**



Treatment Time:

**MAXIMUM
TREATMENT TIME IS
45 MINUTES.**

Do not exceed treatment time

Clients can use this 1x a week, up to 4 weeks then every
6-8 weeks after.

Or

1x a month

Your client should always consult a dentist before
beginning a new oral hygiene routine.



SAME STRENGTH



While this gel is technically the same "strength" as your blended gel, being only hydrogen peroxide (and not a blend of carbamide and hydrogen) it may work better on clients with deeper stains or poor enamel. Due to its concentration, it may also last longer.

SENSITIVITY




The chances of sensitivity are much higher with this gel. Hydrogen peroxide is much more acidic with a higher pH balance.

DEEPSCRUBS & STAIN SHIELDS



Because of its concentration, deep scrubs are highly recommended to keep the process as comfortable as possible for your clients. Remineralizing the teeth during and after treatment is important.

**IMPORTANT
DETAILS**

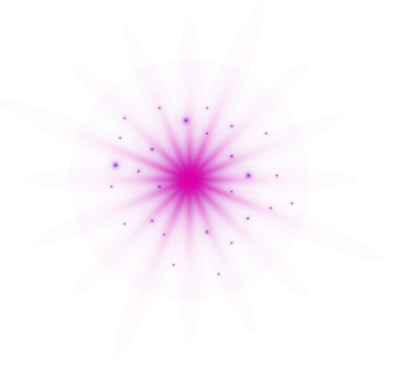

FOR PROFESSIONAL USE ONLY



16% Hydrogen Peroxide
Teeth Whitening Gel is

**Not
recommended**

for use on caps, fillings, veneers or dentures etc.

It is not recommended for pregnant or nursing women. Must be 18+ years of age.

Do not send home with your client. For professional, chairside treatment only.

Why remineralize?

Teeth whitening is an oxidization process and is naturally drying to the tooth (during treatment and up to 24 hours post treatment). By using a product with a higher pH balance, the risk of sensitivity also increases. Rehydrating the enamel during treatment (deep scrubs) helps keep teeth hydrated and lower the risk of sensitivity

Remineralizing Gel...

...such as the stain shield pens help continuously replenish teeth enamel with essential minerals and hydration, prolonging results and helping with sensitivity.



Tingling, Pain or sensations

can be common.



16% Hydrogen Peroxide may cause tingling on the lips, gums and tongue. This gel has a strong HP taste. Extra care to apply vitamin E oil is important.


Guarantees

PEROXIDE & RESULTS

While your client will most likely see results since this is a bleaching gel, we have opted not to advertise its guarantee due to its risk of sensitivity. Since the possibility of stopping treatment may interfere with the delivery of results, we do not recommend making these promises for this product.

SENSITIVITY

Unfortunately is expected and chances are 50/50. This product is powerful and if your client does not have issues should have amazing results.

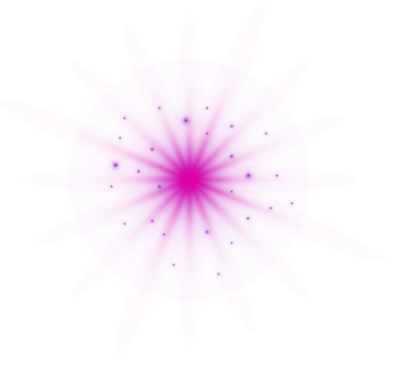

Must be
18+ years
of age.

WAIVER

A seperate waiver

IS AVAILABLE IN YOU RESOURCE CENTRE.
REVIEW THIS CAREFULLY AS THERE ARE
CHANGES AND AREAS FOR YOU/YOUR
BUSINESS TO PERSONALIZE.



Let us know

WHAT YOU THINK! ARE YOU INTERESTED IN
MORE GELS BEING BROUGHT IN?
POSSIBLE 25% HP GELS (THESE WILL
REQUIRE GINGIVAL BARRIERS)?

KEEP BUILDING YOUR EMPIRES! WE'RE
EXCITED TO OFFER YOU ANOTHER WAY TO
EXPAND YOUR MENU AND ARE HERE TO HELP
IN ANY WAY WE CAN!